

SPECIFICATIONS

Q MASTER

Laser Type		Nd:YAG Laser
Wavelength		532, 1064nm
Pulse Duration	532/1064	5-6ns
	1064 Genesis	330 μ s
Output Power	532	50-500mJ
	1064	100-1800mJ
	1064 Genesis	100-2300mJ
	585	50-250mJ
650		50-150mJ
Spot Size		2-10mm
Frequency		1-10Hz
Power Supply		220-240V~, 50/60Hz
Cooling System		Water Cooling
Dimensions		450(W) x 947(D) x 920(H)mm
Weight		95kgs
Display		10.1" Touch Screen Color LCD

Q MASTER PLUS

Laser Type		Nd:YAG Laser
Wavelength		532, 1064nm
Pulse Duration	532/1064	6-8ns
	1064 Genesis	330 μ s, 400 μ s, 600 μ s
	1064 Long	5-30ms
Output Power	532	50-500mJ
	1064	100-1800mJ
	1064 Genesis	100-4500mJ
	1064 Long	1-50J
	585	50-250mJ
	650	50-150mJ
Spot Size		2-15mm
Frequency		1-10Hz(Normal), 0.5-2.0Hz(Long)
Power Supply		220-240V~, 50/60Hz
Cooling System		Water Cooling
Dimensions		450(W) x 947(D) x 920(H)mm
Weight		95kgs
Display		10.1" Touch Screen Color LCD




Q MASTER PLUS



Q-Switched Nd:YAG Laser
Q-Switched & Long Pulsed Nd:YAG Laser

Introduction

Q-master irradiates laser of single wavelength – 1064nm or 532nm – toward the skin for a very short time (nano second level) and transforms optical energy into thermal energy. Using the selective photothermolysis and heat resolving principle, it protects the normal skin but destroys and treats the pigment lesions. In addition, it absorbs tattoo ink by resolving the target pigment into fine particles. As shown in the figures right, the laser of 1064nm penetrates deep into the skin, while the laser of 532nm and 1064nm absorbs red oxyhemoglobin and reddish brown melanin respectively.

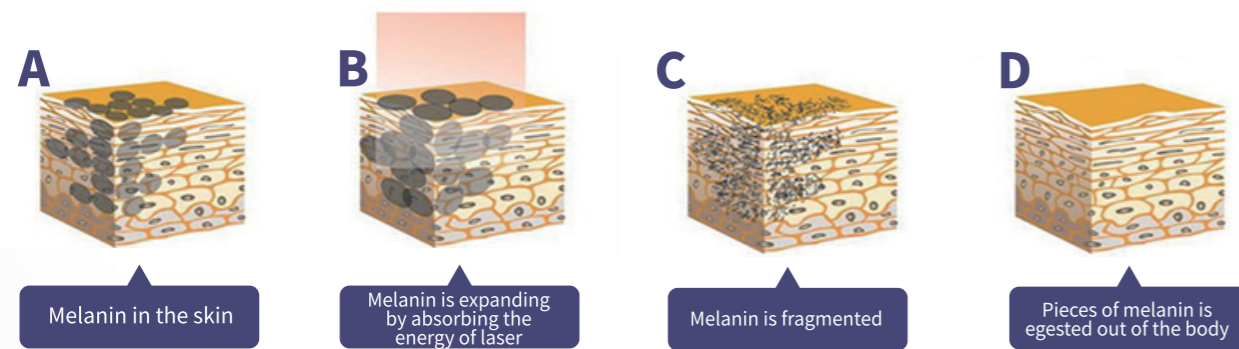


What is the Selective Photothermolysis?

A principle under which the normal skin can be protected while the pigment areas are absorbed and destroyed by the laser selectively.

Top-Hat Mode

Top Hat Mode Beam provides that evenly energy can be evenly irradiated at targeted skin area. It makes proper treatment and can minimize skin damage

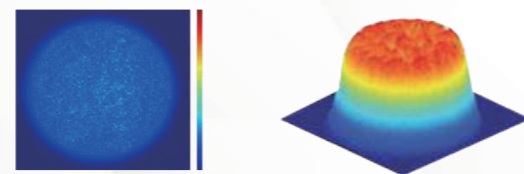

Features

- Four Wavelength available [532nm, 1064nm, 585nm(optional) and 650nm(optional)]
- Safe, No side effect & Faster treatment
- Flat-Top-Mode Applied
- (New concept of BMO method by optimizing beam mode)

Q MASTER

Q-Switched Nd:YAG Laser

- Adjustable Spot Size 2-10mm
- Highly effective against pigmentation

Q MASTER PLUS

Q-Switched Nd:YAG Laser & Long Pulsed Nd:YAG Laser (2 in 1)

- Adjustable Spot Size 2-10mm & 11-15mm (Two Hand pieces)
- Highly effective for treating pigmentation and Hair removal
- One Device can treat for both of skin care and hair removal

Applications

- Tattoo Removal
- Toning
- Fine Wrinkle
- Nevus of Ota
- Cafe Au Lait
- Hair Removal
- Pigmentation
- Carbon Peel
- Lentigo

Before & After

